




# Masimo SafetyNet™

Radius PPG™ & RadiusT°™

## Daily Care & Activities

**Warning:** Do not self-diagnose or self-medicate on the basis of the measurements. Always consult your doctor.


Before beginning the **Daily Care & Activities**, complete sensor and app setup by following the **Setup Instructions** included in your kit.

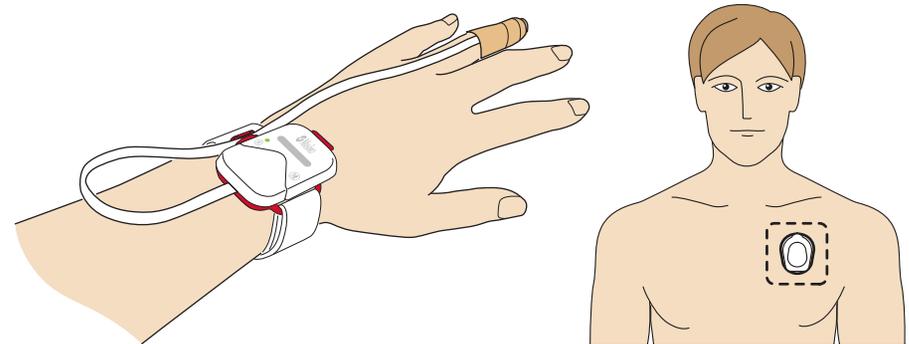

## Daily Tasks on Phone


Example shown – Your screens may differ

### Begin CareProgram™

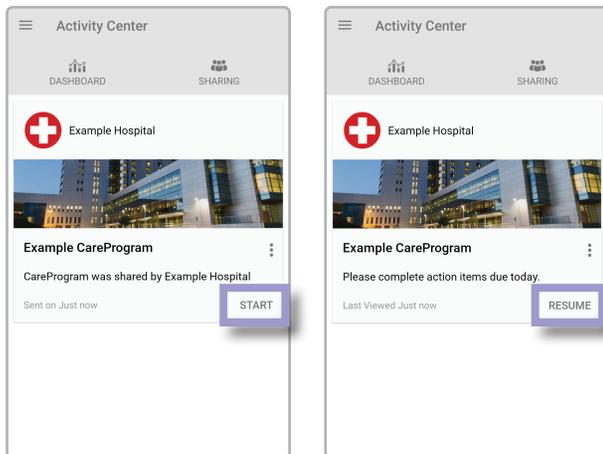

Select "Start" to begin your CareProgram or "Resume" to continue if you started it on a previous day.


### Answer Questions

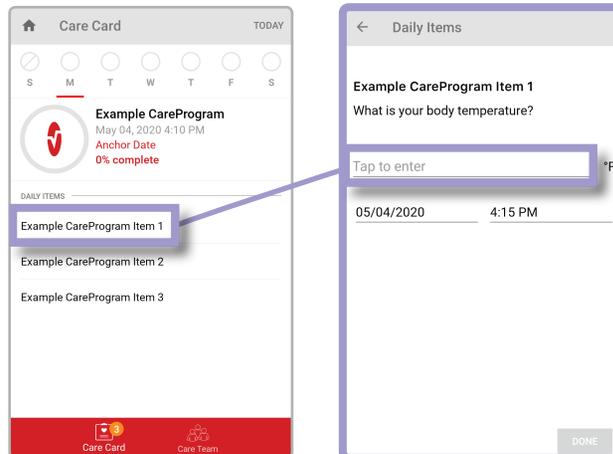

Select each item listed under "Daily Items" to answer question(s) from your healthcare provider.

Answer the question that pops up. Repeat until you've completed all "Daily Items" for each day.

### Check Connection

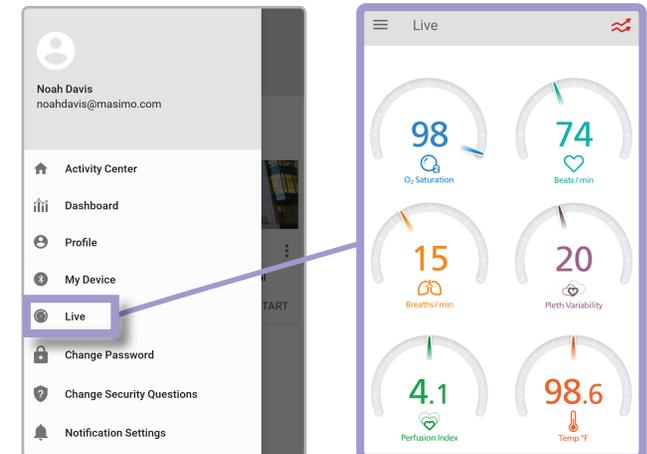


From Activity Center, tap the menu icon to open the menu. Tap "Live" to view your live data from your sensor.


If your sensor is applied correctly and connected, you will see your live data with numbers.

# Caring for Your Sensors


## Change Finger Sensor Tape

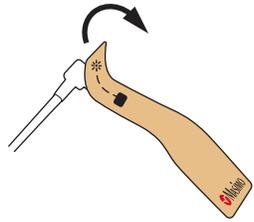


Remove the old tape leaving sensor in place

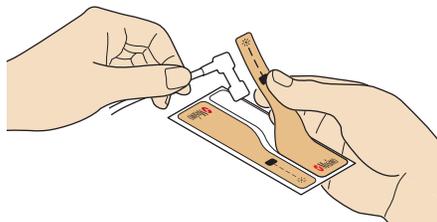

Peel new tape from backing and attach to sensor

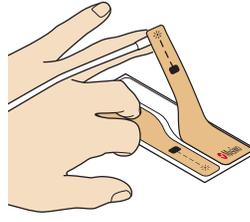



Attach sensor to finger

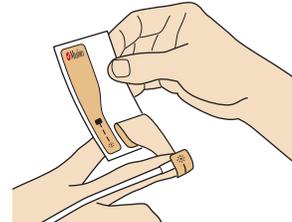

Peel the remaining tape off backing


## Check Sensor Battery

If battery light is orange or red, it's time to change your sensor.

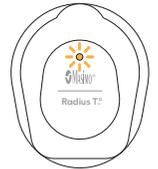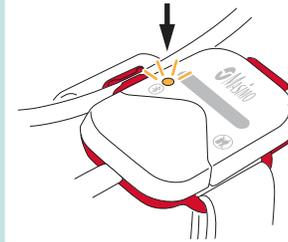

## Changing Your Finger Sensor


1. First remove the sensor from your wrist and finger
2. Remove the chip from your old sensor by pressing down on the red tab
3. Follow the **"How to Apply Your Sensor"** section on the **Setup Instructions** included in your kit

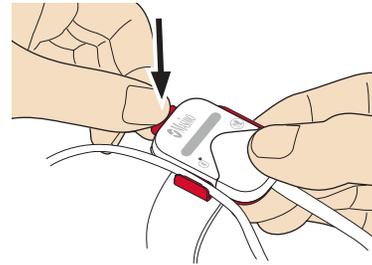

Save the Chip!  
You will use it again shortly.

## Re-position Chest Sensor

If needed, gently wipe the exposed adhesive with an alcohol swab and allow to dry before reapplying.

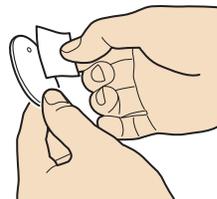

Sensor can be worn in the shower but should not be submerged.



## Other Sensor Tips

- Remove the finger sensor when you shower but you can leave it on when you wash your hands.
- Replace finger sensor tape once a day. You can change it more often if the tape gets dirty, wet, or no longer sticks.
- Change the sensor if battery light is orange or red.
- For additional resources, including videos and instructions, visit [mymasimo.com](https://www.mymasimo.com)